

Be sure to read this instruction manual before using this product.
Keep this manual available for reference when needed.

Instructions for Use

Trade Name: Malleable Suction Tube

Warning

1. Please read these instructions carefully prior to using this product. Product should be used according to these instructions and pay close attention to the safety of patients. Failure to follow manufacturer's recommendations may cause harm or injury to patient.

For the US market

Do not reuse the device when it is used on a patient with or suspected of having Creutzfeldt-Jakob Disease (CJD) or variant CJD (vCJD).

For the market outside the US

When the device is used on a patient with or suspected of having CJD or vCJD, make sure to adhere to the most recent and updated restrictions available in each country and/or region for its reuse. Be cautious of the possibility of secondary infection. Refer to www.a-k-i.org or AAMI Standard ST79 for more information related to cleaning and sterilization.


2. Do not bend the device more than 30 degrees.
[The device may break, and fragments may remain in the patient. Also, it may become difficult to insert the mandolin.]

3. Do not repeatedly bend the device at the same location.
[The device may break, and fragments may remain in the patient.]

Contraindication / Prohibition

1. Use for intended purpose only
Use devices for their intended purposes only. Incorrect use could cause this product to break.
2. Prohibition of use of chemicals
Avoid exposing this product to chemicals. Doing so could damage this product due to corrosion.
3. Prohibition of secondary processing of this product
Do not apply any secondary processing to this product. For example, do not apply impacts or vibration markings to the surface of this product. Doing so could break this product.
4. Handle with care
Handle this product with care, as it can be deformed or damaged. Rough handling could significantly reduce the service life of devices and appliances.
6. Prohibition of use of polishing powder and wire wool
When cleaning this product, do not attempt to polish its surfaces with rough polishing powder or wire wool. This could cause scratches on the surface of this product and result in rust or corrosion
7. Prohibition of use of household detergents
Use only medical detergents to clean this product. Do not use any household detergent. Washing this product with an improper detergent could result in discoloration or corrosion.

Shape / Structure

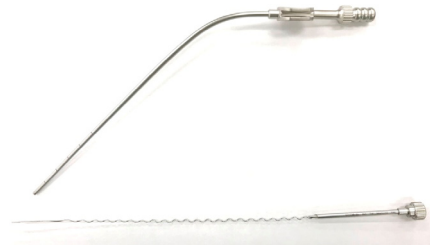

Code No.	Product Description
07-794-91	Malleable Suction Tube, 6Fr, Str, SUS
07-794-92	Malleable Suction Tube, 6Fr, Cvd, Up, SUS
07-794-93	Malleable Suction Tube, 6Fr, Cvd, Down, SUS
07-794-94	Malleable Suction Tube, 9Fr, Str, SUS
07-794-95	Malleable Suction Tube, 9Fr, Cvd, Up, SUS
07-794-96	Malleable Suction Tube, 9Fr, Cvd, Down, SUS
07-794-97	Malleable Suction Tube, 12Fr, Str, SUS

Material: Stainless Steel

Intended Purpose

A device that attaches to a suction instrument to control and direct suction during a surgical procedure or treatment. This device is a general-purpose suction tip and is reusable.

Instructions for use

- Before using this product, inspect, wash, and sterilize in accordance with these instructions
- Only use your hands when bending the pipe section.
- Bend the device within the range (50 mm) marked on the tip.
- Inspect the instruments for damage and cracks prior to and after use. (Refer to Maintenance / Inspection)

CAUTIONS & WARNINGS

1. **WARNING**
Device must be sterilized by users under the standard sterilization conditions or the validated sterilization conditions which validity is proven by medical organizations in each country or region.
2. Defect/Adverse event
Defect
 - Corrosion and pitting caused by use of chemicals
 - Damage or breakage caused by the corrosion or pitting
 Adverse events
 - Broken pieces of metal from the damaged instrument falling into the patient.

Storage / Life

1. Please store the device in normal ambient temperature areas. Do not store in areas of high humidity where the temperature may dramatically vary causing condensation. Do not store on or near chemicals as the chemicals may cause damage to the device.
2. Service life of this product: 1 year
(Subject to following manufacturer's specified maintenance, inspection and proper storage requirements.)

Maintenance / Inspection

1. Check prior to each use
 - 1.1 Operational and functional checks
Conduct daily and pre-operation checks of this product to make sure that it functions properly.
 - 1.2 Inspect the instrument for damage, fractures or cracks.



Examples of damage

2. Check after each use
 - 2.1 Immediately wash with clean water
If exposed to bleach or antiseptic solutions, immediately wash: Wash and rinse with clean water immediately and immerse in neutral enzyme detergent to remove any bleach or antiseptic solution, which may contain chlorine or iodine and can damage the instrument. Manually remove contaminated matter by hand or with an ultrasonic-cleaner.
 - 2.2 Further remove any remaining contamination with a brush.
 - 2.3 Select a proper detergent for each decontamination method and maintain appropriate density and handling. Use of a neutral detergent is recommended.
 - 2.4 Use a soft towel, a plastic brush, or a water gun for cleaning.
 - 2.5 Do not use a metal brush, coarse polishing agents, or apply excessive force when handling the device.
 - 2.6 Reverse osmosis water is recommended to wash this product.
 - 2.7 Only use reverse osmosis water for the final rinse.
 - 2.8 It is recommended to use a washer-disinfector for this device.
Thermal Disinfection can be used by following the manufacturer's defined parameters.: Thermal Disinfection Band: 90-93 °C/194.0-199.4°F, 5-10 minutes (A0 value: 3000-12000) (reference EN ISO 15883-1)
 - 2.9 Fully dry this product immediately after washing it.
Fully dry this product immediately after washing it. Do not leave it wet for a longer time than necessary as residual water may damage the instrument.
 - 2.10 Use distilled or deionized water.
Use distilled or deionized water to wash this product. Residual chlorine and organic matters from standard tap water may cause stains and may cause rusting.
 - 2.11 Use a water-based anticorrosive lubricant
Lubricating oil is completely removed by washing. Do not use without lubricant oil to sliding part, or galling could occur. After washing this product, apply a water-based anticorrosive lubricant prior to sterilization.
3. Sterilization
Device must be sterilized by users in accordance with validated sterilization procedures, such as an autoclave, that are regulated by medical organizations in each country or region. Do not use hydrogen peroxide gas plasma sterilization. Sterilization of the device may be accomplished by steam. The recommended sterilization parameters are as follows,

ISO/TS17665-2	
Temp.	Minimum exposure time
121°C / 249.8°F	15 Min
126°C / 258.8°F	10 Min
134°C / 273.2°F	3 Min

For the US market

FDA has not approved or cleared medical devices, including sterilizers, for the intended use of reducing the infectivity of TSE agents (i.e., prions).

For the market outside the US

When the device is used on a patient with or suspected of having CJD or vCJD, make sure to adhere to the most recent and updated restrictions available in each country and/or region for its reuse.

Maintenance and check by agents

For safe use of this instrument, inspect device prior to and after each use. Alternative or no review and/or inspection may cause injury to the patient and/or healthcare worker and may decrease the performance and function of this device. Additionally, it is recommended to schedule a periodic inspection through your local distributor or the manufacturer.

Packing

1 piece per pack

Matters related to warranty period

MIZUHO Corporation will repair defective parts of this product without charge for one year from the date of delivery/installment except for cases of damage caused by a third party's repair, act of nature, improper use or intentional damage. All other warranty terms and conditions are subject to regulations of MIZUHO Corporation.

Name and address of manufacturer



MIZUHO Corporation
3-30-13 Hongo, Bunkyo-ku, Tokyo 113-0033, Japan
<http://www.mizuho.co.jp>