


Trade Name: Clip Applier
(SUGITA CLIP-APPLYING FORCEPS)

Straight Angled15° Angled30° Side angled

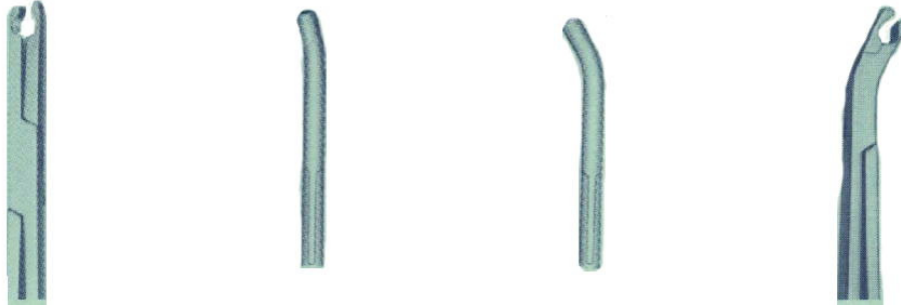


TYPE		Cat No.	Overall Length	Specific Clips	
STANDARD·LARGE	Straight	20.0cm	07-941-01	20.0cm	Sugita Aneurysm Clips-STANDARD type, LARGE, FENESTRATED LARGE and TEMPORARY STD.
STANDARD·LARGE	Angled 15°	20.0cm	07-941-02	20.0cm	
STANDARD·LARGE	Angled 30°	20.0cm	07-941-03	20.0cm	
STANDARD·LARGE	Side angled	21.0cm	07-941-04	21.0cm	
STANDARD·LARGE	Straight	21.5cm	07-941-60	21.5cm	Sugita Aneurysm Clips-MINI type and TEMPORARY MINI ONLY.
STANDARD·LARGE	Angled 30°	21.5cm	07-941-61	21.5cm	
MINI	Straight	20.0cm	07-942-01	20.0cm	
MINI	Angled 30°	20.0cm	07-942-02	20.0cm	
MINI	Straight	21.5cm	07-942-62	21.5cm	
MINI	Angled 30°	21.5cm	07-942-63	21.5cm	Sugita Aneurysm Clips-LONG type
MINI	Side angled	21.0cm	07-942-64	21.0cm	
LONG	Straight	20.0cm	07-948-10	20.0cm	

MATERIAL:.....Jaws of Co-Cr alloy (ASTM F1058 G2,ISO 5832/7)
Other parts stainless steel (AISI 410, 420)

INSTRUCTION FOR USE:

- (Fig.1).....It is most important that the clip is properly located within grooves of the forceps jaws as illustrated.
- (Fig.2).....Under no circumstances must the clip be held in an angled position since improper opening of the clip will result and the spring portion may be damaged.
- (Fig.3).....If the clip is not fully located within the jaws of the forceps it may slip or springout and the spring portion may be damaged.

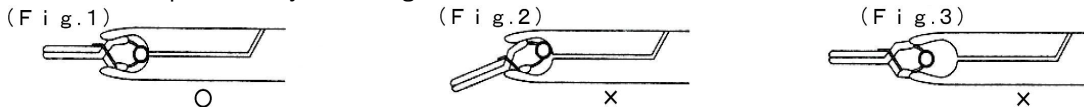

CAUTION:.....SUGITA CLIP APPLYING FORCEPS are designed for use with Sugita clips only.
Use only with the Sugita clips specified.
Do not use with other manufacturers clips.
SUGITA CLIP APPLYING FORCEPS must be inspected before each use, if damaged do not use, Defective applying forceps can result in damage to the spring mechanism of a clip and alter the holding force.

* **Name and address of manufacturer**

MIZUHO Corporation
3-30-13 Hongo, Bunkyo-ku, Tokyo 113-0033, Japan
<http://www.mizuho.co.jp>

* **Authorized Representative Europe**

Emergo Europe B.V.
Prinsessegracht 20, 2514 AP, The Hague
The Netherlands