


Trade Name: Clip Applier
(SUGITA Keyhole Applier - 360° Rotation -)

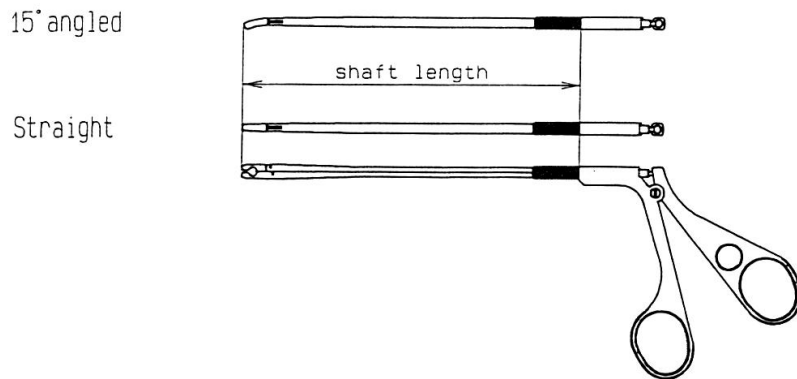


TYPE	Cat No.	Shaft Length	Overall Length	Specific Clips
STANDARD·LARGE Straight	07-941-10	14.0cm	23.2cm	Sugita Aneurysm Clips -STANDARD LARGE, FENESTRATED LARGE and TEMPORARY STD type.
STANDARD·LARGE 15° Angled	07-941-11	14.0cm	23.2cm	
STANDARD·LARGE Straight	07-941-20	13.0cm	22.2cm	
STANDARD·LARGE 15° Angled	07-941-21	13.0cm	22.2cm	
MINI Straight	07-942-10	14.0cm	23.2cm	Sugita Aneurysm Clips -MINI and TEMPORARY MINI type ONLY.
MINI 15° Angled	07-942-11	14.0cm	23.2cm	
MINI Straight	07-942-20	13.0cm	22.2cm	
MINI 15° Angled	07-942-21	13.0cm	22.2cm	

MATERIAL:.....Jaws of Co-Cr alloy (ASTM F1058 G2,ISO 5832/7)
Other parts stainless steel (AISI 431, 420)

INSTRUCTION FOR USE:

- (Fig.1).....It is most important that the clip is properly located within grooves of the applier jaws as illustrated.
- (Fig.2).....Under no circumstances must the clip be held in an angled position since improper opening of the clip will result and the spring portion may be damaged
- (Fig.3).....If the clip is not fully located within the jaws of the applier it may slip or spring out and the spring portion may be damaged.

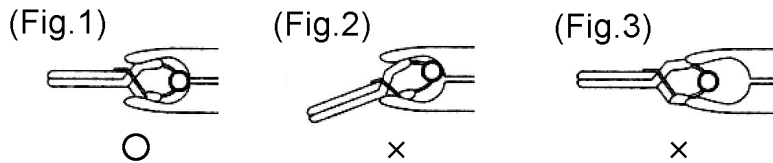

CAUTION:.....SUGITA Keyhole Applier is designed for use with Sugita clips only.
Use only with the Sugita clips specified.
Do not use with other manufactures clips.
SUGITA Keyhole Applier must be inspected before each use, if damaged do not use, Defective applying forceps can result in damage to the spring mechanism of a clip and alter the holding force.

*** Name and address of manufacturer**



MIZUHO Corporation
3-30-13 Hongo, Bunkyo-ku, Tokyo 113-0033, Japan
<http://www.mizuho.co.jp>

*** Authorized Representative Europe**



Emergo Europe B.V.
Prinsessegracht 20, 2514 AP, The Hague
The Netherlands