


Trade Name: Clip Applier  
(SANO Double Action Clip Applier)

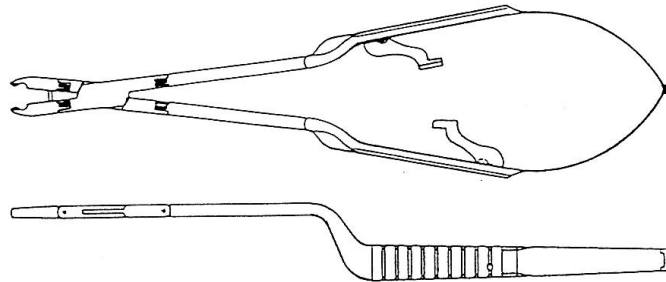


TYPE	Cat No.	Overall Length	Specific Clips
Double Action STANDARD·LARGE	07-946-00	20.0cm	Sugita Aneurysm Clips-STANDARD, LARGE, FENESTRATED LARGE, TEMPORARY STD and BOOSTER type.
Double Action MINI	07-950-12	20.0cm	Sugita Aneurysm Clips-MINI and TEMPORARY MINI type ONLY.

**MATERIAL:**.....Jaws of Co-Cr alloy (ASTM F1058 G2,ISO 5832/7)  
Other parts stainless steel (AISI 410,420)

**GEOMETRY:**.....Double Action-The jaws can be adjusted to many angles and configurations using the adjusting wrench provided.

**INSTRUCTION FOR USE:**

- (Fig.1).....It is recommendable to hold clip after aligning the jaws with the specialized angle-adjustment wrench, and simultaneously adjusting the angle.  
In case of adjusting the angle after holding clip with the applier, this specialized angle-adjustment wrench is also available.
- (Fig.2).....It is most important that the clip is properly located within grooves of the applier jaws as illustrated.
- (Fig.3).....Under no circumstances must the clip be held in an angled position since improper opening of the clip will result and the spring portion may be damaged
- (Fig.4).....If the clip is not fully located within the jaws of the applier it may slip or spring out and the spring portion may be damaged.

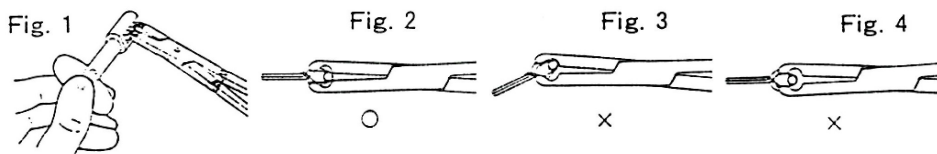

**CAUTION:**.....SANO Double Action Clip Appliers are designed for use with Sugita clips only.  
Use only with the Sugita clips specified.  
Do not use with other manufactures clips.  
SANO Double Action Clip Appliers must be inspected before each use, if damaged do not use.  
Defective applying forceps can result in damage to the spring mechanism of a clip and alter the holding force.

\* Name and address of manufacturer



MIZUHO Corporation  
3-30-13 Hongo, Bunkyo-ku, Tokyo 113-0033, Japan  
<http://www.mizuho.co.jp>

\* Authorized Representative Europe



Emergo Europe B.V.  
Prinsessegracht 20, 2514 AP, The Hague  
The Netherlands