


Trade Name: Clip Applier
(SANO All-Angle Clip Applier)

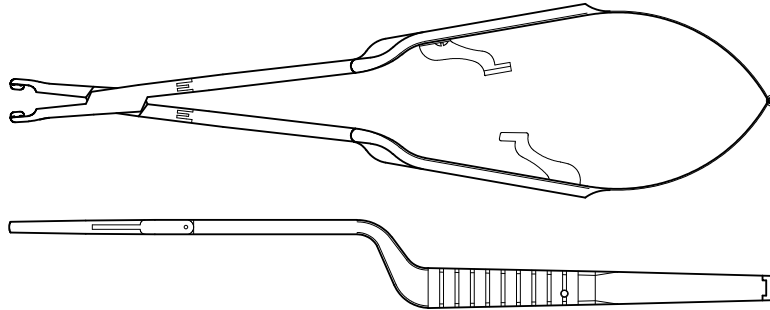

TYPE	Cat No.	Overall Length	Specific Clips
All-Angle STANDARD • LARGE	07-947-00	20.0cm	Sugita Aneurysm Clips-STANDARD, LARGE, FENESTRATED LARGE, TEMPORARY STD and BOOSTER type.
All-Angle MINI	07-950-13	20.0cm	Sugita Aneurysm Clips-MINI and TEMPORARY MINI type ONLY.

MATERIAL:.....Jaws of Co-Cr alloy (ASTM F1058 G2,ISO 5832/7)
Other parts stainless steel(AISI 410,420)

GEOMETRY:.....All-Angle-The jaws can be rotated through 220 ° and angled using the adjusting plate provided.

INSTRUCTION FOR USE:

- (Fig.1).....It is recommendable to hold clip with the applier after setting the slot-angle of socket correctly and right-left symmetrically.
The slot-angle can be adjusted right-left symmetrically by holding the adjusting plate with the applier, as per Fig.1.
- (Fig.2).....It is most important that the clip is properly located within grooves of the applier jaws as illustrated.
- (Fig.3).....Under no circumstances must the clip be held in an angled position since improper opening of the clip will result and the spring portion may be damaged
- (Fig.4).....If the clip is not fully located within the jaws of the applier it may slip or spring out and the spring portion may be damaged.

CAUTION:.....SANO All-Angle Clip Appliers are designed for use with Sugita clips only.
Use only with the Sugita clips specified.
Do not use with other manufactures clips.
SANO All-Angle Clip Appliers must be inspected before each use,
if damaged do not use.
Defective applying forceps can result in damage to the spring mechanism of a clip and alter the holding force.

* Name and address of manufacturer



MIZUHO Corporation
3-30-13 Hongo, Bunkyo-ku, Tokyo 113-0033, Japan
<http://www.mizuho.co.jp>

* Authorized Representative Europe



Emergo Europe B.V.
Prinsessegracht 20, 2514 AP, The Hague
The Netherlands