

Instruction for Use


Trade Name: Anesthesia screen L type with clamp

Contraindication/Prohibition

1. This product is only validated for use with Mizuho operating tables.
2. Ensure all clamp handles are tight. Failure to do so may result in a patient injury.

Shape/Structure

1. Shape/Structure

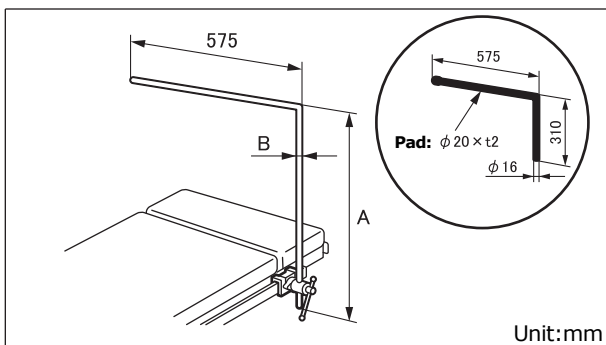

Code No.	Product Description	A (mm)	B (mm)	Weight (kg)
08-062-00	Anesthesia screen L type with clamp	650	16	2.0
* 08-062-00-R1	Anesthesia screen L type with clamp (R1)	650	16	2.0
* 08-062-00-R2	Anesthesia screen L type with clamp (R2)	650	16	2.0

Code No.	Product Description	Insertion hole dia. (mm)	Weight (kg)
08-110-01	Clamp for round bar	16	0.9
* 08-117-09	Clamp for round bar (R1)	16	0.9
* 08-117-11	Clamp for round bar (R2)	16	0.9


2. Material

Main body : Stainless steel rod
Rail clamp : Stainless cast steel

Intended purpose

This is an accessory to an operating table. This is an L-shaped anesthesia screen frame. This frame is to be attached to an angle-adjustable clamp mounted on one of the side rails of an operating table.

Instructions for use

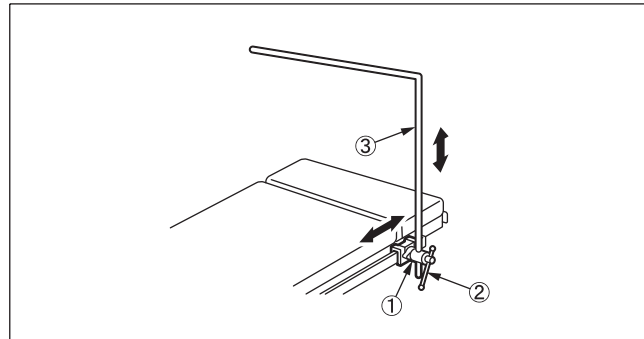

- ①Rail clamp ②Clamping handle
③Anesthesia screen frame

1. Attach the clamp for round bar① to the side rail of an operating table.
2. Insert the anesthesia screen frame③ into the rail clamp① and tighten the clamping handle② to temporarily secure the frame.
3. Move the anesthesia screen frame③ to a position appropriate for the patient's posture.
4. After the adjustment, tighten the clamping handle② to secure the clamp① and the anesthesia screen frame③.

Caution

1. Important caution

When working with an operating table, take care not to allow this product to make contact or interfere with the table top or with other tools and appliances used in combination with this product.

Storage/Life

1. Do not store the device in high temperatures or in areas with high humidity where the temperature has drastic variations.
2. Service life of this product: Seven (7) years (subject to the specified maintenance and inspection and is stored properly)

Maintenance/Inspection

1. Check that this product is not damaged or broken before and after use.
2. If you need to have this product repaired, consult your local dealer or Mizuho.

Packing

1 piece per pack

Matters related to warranty period

MIZUHO Corporation will repair defective parts of this product without charge for one year from the date of delivery/installment except for cases of damage caused by a third party's repair, act of nature, improper use or damage on purpose. All other warranty terms and conditions are subject to regulations of MIZUHO Corporation

**** Name and address of manufacturer**



MIZUHO Corporation
3-30-13 Hongo, Bunkyo-ku, Tokyo 113-0033, Japan
<http://www.mizuho.co.jp>

**** Authorized Representative in Europe**



Emergo Europe B.V.
Prinsessegracht 20, 2514 AP, The Hague
The Netherlands