

## Instruction for Use

### Trade Name: Narrow Head Section

#### Warning

When using operating table accessories, always pay close attention to the patient. Patients left in the same posture for an extended period of time are at risk for developing bedsores.


#### Contraindication/Prohibition

1. This product is only validated for use with Mizuho operating tables.
2. Ensure all clamp handles are tight. Failure to do so may result in a patient injury.

#### Shape/Structure

1. External view

Unit: mm

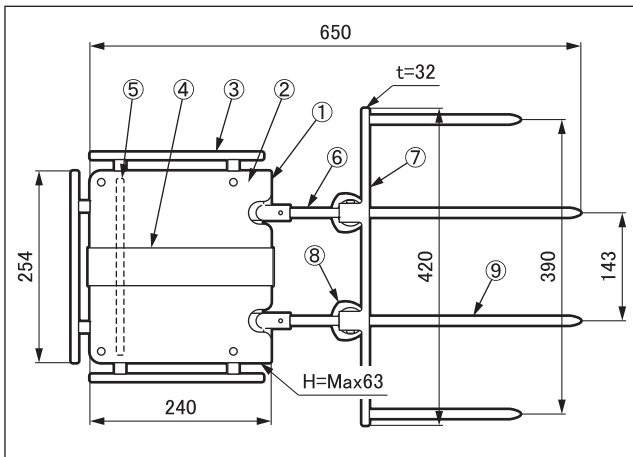

Code No.	Product Description
08-075-00-R1	Narrow Head Section (R1)
08-075-00-R2	Narrow Head Section (R2)

Note: Size of side rail③ is different on R1 and R2.

2. Weight

Headrest: 5.5 kg

Attachment: 2.0 kg

3. Material

- ① Headrest frame: Aluminum alloy casting
- ② Board: Thermosetting resin laminated board
- ③ Side rail: Stainless steel bar
- ④ Attachment tape: Nylon
- ⑤ Headrest refraction lever: Aluminum alloy bar
- ⑥ Headrest insert shaft: Stainless steel bar
- ⑦ Attachment frame: Stainless steel bar
- ⑧ Fixing handle: Stainless steel bar
- ⑨ Attachment insert shaft: Stainless steel bar

Note: The number corresponds to those used in 1. External view.

4. Function


Headrest refracting range:

Upwards: 60°, Downwards: 90°(Manual)

#### Intended purpose

Accessory attached to the surgical table. Narrow width headrest. It is attached to the main unit of surgical table via the attachment, to support the head.

#### Instructions for use

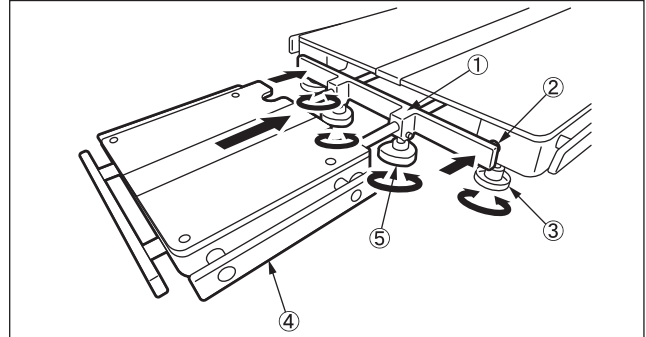

1. Remove the head plate from the surgical table.
2. Insert the attachment ① in the back plate insert hole ②, and fix it by turning the head plate fixing handle ③.
3. Insert the narrow head section ④ into the attachment ①, and fix it by turning the fixing handle ⑤.

#### Caution

1. Warning

Check always if tightening sections are fixed securely. Otherwise the apparatus may be disengaged or moved, causing hazards on the patient.

2. Important precautions

When operating the surgical table, take care to avoid contact or interference with other accessories or the table top.

#### Storage and Shelf Life

1. Do not store the device in high temperatures or in areas with high humidity where the temperature has drastic variations.
2. Service life of this product: 7 years  
(subject to the specified maintenance and inspection and is stored properly)

#### Maintenance/Inspection

1. Check that this product is not damaged or broken before and after use.
2. Cleaning and disinfection  
Wipe off blood, chemicals, contaminants, and other stains with water, and clean the device with gauze or other material moistened with alcohol to disinfect.
3. In case of a malfunction  
If you need to have this product repaired, clearly label the device as "Broken", "Need repair" etc. and contact your local dealer or Mizuho.

<By suppliers>

Maintenance and inspection can only be carried out certified suppliers of Mizuho.

#### Packing

1 piece per pack

#### Matters related to warranty period

MIZUHO Corporation will repair defective parts of this product without charge for one year from the date of delivery/installment except for cases of damage caused by a third party's repair, act of nature, improper use or damage on purpose. All other warranty terms and conditions are subject to regulations of MIZUHO Corporation.

**\*Name and address of manufacturer**



MIZUHO Corporation  
3-30-13 Hongo, Bunkyo-ku, Tokyo 113-0033, Japan  
<http://www.mizuho.co.jp>

**\*Authorized Representative Europe**



Emergo Europe B.V.  
Prinsessegracht 20, 2514 AP, The Hague  
The Netherlands