

MES-CK08-079-00EN-0

PREPARED: (2017-11-17) (Version 2)\*  
(2017-01-25) (Version 1)

## Instruction for Use

### Trade Name: Wrist Strap

#### Warning


When using operating table accessories, always pay close attention to the patient. Patients left in the same posture for an extended period of time are at risk for developing bedsores.

#### Contraindication/Prohibition

1. This product is only validated for use with Mizuho operating tables.
2. Do not wrap the support around a patient too tightly. Failure to do so may result in a patient injury.
3. Do not apply excessive load on the on the leaf spring. Failure to do so may result in breakage.

#### Shape/Structure

##### 1. External view

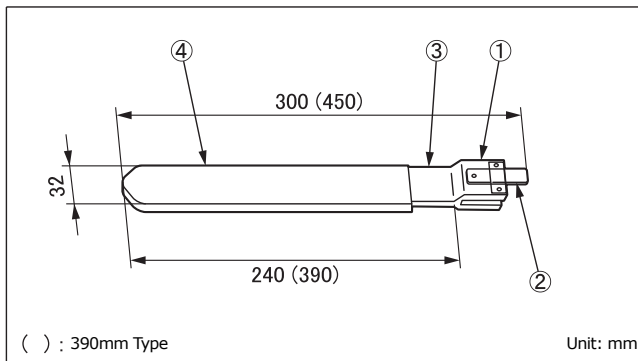

Code No.	Product Description	Weights
08-079-00	Wrist Strap (240 mm)	0.3kg
08-079-00-R1	Wrist Strap (240 mm) (R1)	0.3kg
08-079-00-R2	Wrist Strap (240 mm) (R2)	0.3kg
08-079-01	Wrist Strap (390 mm)	0.6kg
08-079-01-R1	Wrist Strap (390 mm) (R1)	0.6kg
08-079-01-R2	Wrist Strap (390 mm) (R2)	0.6kg

Note: Clamp ① needs to be selected depending on the rail size.

##### 2. Material


- ① Clamp: Copper alloy plate
- ② Leaf spring: Coal bed steel
- ③ Band: Copper alloy plate
- ④ Cover: Black leather

Note: Reference numbers are corresponding to that indicated in appearance diagram shown in 1. External view.

#### Intended purpose

This is an accessory to an operating table and is used to position a forearm of a patient.

#### Instructions for use

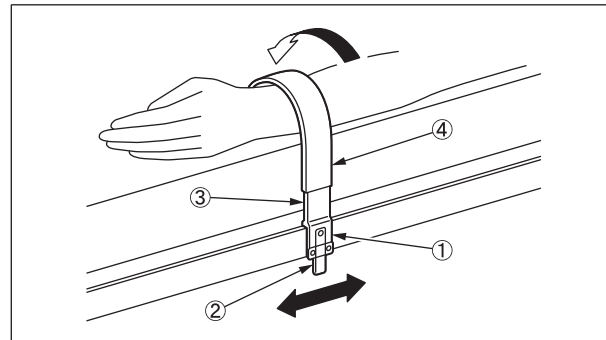

1. Attach the clamp① to the side rail of an operating table.
2. Pull the leaf spring② to adjust the position.
3. Bend the band③ with the cover④ attached as appropriate for the patient's arm.

#### Caution

##### Important caution

When working with an operating table, take care not to allow this product to make contact or interfere with the table top or with other tools and appliances used in combination with this product.

#### Storage and Shelf Life

1. Do not store the device in high temperatures or in areas with high humidity where the temperature has drastic variations.
2. Service life of this product: 7 years (subject to the specified maintenance and inspection and is stored properly)

#### Maintenance/Inspection

1. Check that this product is not damaged or broken before and after use.
2. Cleaning and disinfection  
Wipe off blood, chemicals, contaminants, and other stains with water, and clean the device with gauze or other material moistened with alcohol to disinfect.
3. In case of a malfunction  
If you need to have this product repaired, clearly label the device as "Broken", "Need repair" etc. and contact your local dealer or Mizuho.

<By suppliers>

Maintenance and inspection can only be carried out certified suppliers of Mizuho.

#### Packing

1 piece per pack

#### Matters related to warranty period

MIZUHO Corporation will repair defective parts of this product without charge for one year from the date of delivery/installment except for cases of damage caused by a third party's repair, act of nature, improper use or damage on purpose. All other warranty terms and conditions are subject to regulations of MIZUHO Corporation.

**\*Name and address of manufacturer**



MIZUHO Corporation  
3-30-13 Hongo, Bunkyo-ku, Tokyo 113-0033, Japan  
<http://www.mizuho.co.jp>

**\*Authorized Representative Europe**



Emergo Europe B.V.  
Prinsessegracht 20, 2514 AP, The Hague  
The Netherlands