Intended Use

This set is intended for use in the surgical setting to hold and protect T2 Applying forceps and Sugita T2 and AVM clips during steam sterilization processes and subsequent storage.

CAUTION

Federal law restricts this device to sale by or on order of a physician.

This IFU applies to Sugita T2 and AVM Clips Organizational Trays. Prior to use of the trays, ensure the products are not damaged and used only by persons with the required training, knowledge and/or experience. No additional training is required as Surgeons have an intimate knowledge of anatomy and physiology, and are trained in the use of similar devices as part of their surgical practice. Read, follow and retain these instructions and use the product only for its intended purpose. Prior to use, remove its packaging, carefully clean either by hand or with the use of a machine prior to sterilization. If the product is found damaged or defective do not use.

Instructions for Use

Before and after using this product, inspect for function and/or damage, wash and sterilize in accordance with these instructions. If any issues are found related to the device do not use and contact Mizuho America Representative.

Warnings

- These are the risk of injury and/or failure of the product is not inspected prior and after use
- Risk of injury when applying the product outside your field of view
- Product should be used according to these instructions and pay close attention to the safety of patients. Failure to follow these and/or the healthcare's requirements may cause harm to the patient and/or healthcare workers
- The product must be inspected, cleaned, and sterilized before each surgical procedure.

Storage/Life

- Service of life of Trays: 7 years
- Always inspect the products between use to confirm proper functioning
- Do not store this device in high temperatures or in areas with high humidity where the temperature has drastic variations.

Contraindication/Precautions

- Improper use of this product during handling, surgical use or reprocessing, for which they are indicated, may result in damage.
- Do not use excessive stress or misuse as it may result in damage to the product.
- Use only medical detergents to clean this device. Washing with an improper detergent could result in discoloration or corrosion.
- This device is non-sterile
- Material: plastic, stainless steel, silicone

Sterilization

- Use a validated, properly maintained steam sterilizer. Always follow instructions of the machine manufacturer.
- Do not overload cases. Overloading may inhibit steam flow, cause excessive drying time, and make the product too heavy to safety handle.
- Do not exceed 140°C (284°F) during sterilization cycle
- Do not stack cases. For effective sterilization trays must have adequate steam circulation around all surfaces.
- The product must be placed upright on shelves in order for proper ventilation.
- Do not place the product on their sides or at vertical angles in chamber.
- The product should be wrapped in two layers of 1-ply polypropylene wrap (KC600) or hospital designated FDA cleared sterilization wrapping materials. Always follow the sterilization wrap manufacturer's instruction prior to sterilization.
- Wrapping materials and the trays are designed to allow air removal, steam penetration/evacuation (drying) and maintain the sterility of the internal items.
- Effective sterilization can be achieved following the steam cycle:

Sterilization Method	Pre-vacuum Steam Sterilization	
	Sterilization temp.	Retention time
Sterilization Condition	Sugita Clips	
	135°C/ 275°F	3min.
	132.2°C/ 270°F	4min.
	AVM Clips	
Minimum Dry Time	121°C/ 250°F	30min
	132°C/270°F	2.3min
Minimum Dry Time	Sugita & AVM	50 min

Maintenance

Decontamination

- Reprocess all devices as soon as they are reasonably practical following use.
- Follow instructions and warnings as issued by manufacturers of any decontaminants, disinfectants, and cleaning agents used. Wherever possible avoid use of mineral acids or harsh, abrasive agents
- Do not use high acid (pH 4.0 or lower) or high alkaline (pH 10 or higher) products for disinfection. Neutral pH detergents 7.0-9.0 are preferred
- Remove gross contaminants with a steady stream of lukewarm/cool water, not to exceed 35°C (95°F). Rinse each device thoroughly. Do not use saline or chlorinated solutions. Only use distilled or deionized water.
- Do not soak devices in hot water, alcohol, disinfectants or antiseptics solution. Do not exceed 2 hours soaking in any solution.
- Do not use steel wool, wire brushes or abrasive detergents

Cleaning & Drying: Automated

- Use only validated washer-disinfector machines and low-foaming, non-ionizing cleaning agents and detergents
- Follow the manufacturer's instructions for use, which includes warnings, concentrations and recommended cycles.
- Load the product carefully
- Remove lids, caddies, and liners to wash with Neutral pH detergents 7.0-9.0 are preferred.
- Ensure that soft, high purified water that is controlled for bacterial endotoxins is used in the final rinse stage
- Devices must be thoroughly dried and all residual moisture must be removed before they are sterilized. Use a soft absorbent towel or cloth to dry external surfaces.

Inspection and Function Testing

- After cleaning, visually inspect all surfaces, for complete removal of soil and fluids. If ANY soil or fluid is still visible, return the device for repeat decontamination.
- Visually inspect and check; all devices for damage.
- Remove for repair or replacement any worn out, fractured, corroded, stained, discolored or damaged product. Failure may cause injury or harm to the patients or users.


Note: If the device is returned to the manufacturer or supplier, the device must be decontaminated and sterilized, otherwise a cleaning charge may apply and delay processing of the repair. Contact Mizuho America via phone (800) 699-CLIP (2547) or email customerservice@mizuho.com to obtain an RGA for return of any damaged items.

Packaging

All devices to be wrapped or packaged following local procedures, in ANSI/AAMI ST 46-1993.

Storage

Non-sterile devices or sterile wrapped devices should be stored in dry, clean conditions at an ambient room temperature.

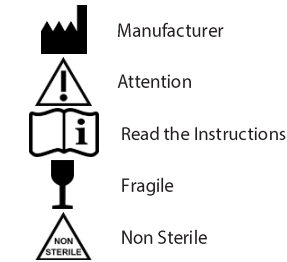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