Intended Use

This set is intended for use in the surgical setting to hold and protect clips during steam sterilization processes and subsequent storage. The removable/replaceable tags allow the user to configure the tray's pockets in any order of the clips, and keep them in this configuration during sterilization, use, and storage. The Mizuho America tray and tags are exclusively compatible with one another and should not be used in conjunction with trays or tags from other manufacturers.

CAUTION

Federal law restricts this device to sale by or on order of a physician.

This IFU applies to Customizable Clip Trays. Prior to use of the trays, ensure the products are not damaged and used only by persons with the required training, knowledge and/or experience. No additional training is required as surgeons have an intimate knowledge of anatomy and physiology, and are trained in the use of similar devices as part of their surgical practice. Read, follow and retain these instructions and use the product only for its intended purpose. Prior to use, remove its packaging, carefully clean either by hand or with the use of a machine prior to sterilization. If the product is found damaged or defective, do not use.

Instructions for Use

Before and after using this product, inspect for function and/or damage, wash and sterilize in accordance with these instructions. If any issues are found related to the device, do not use, and contact Mizuho America Representative.

Warnings

- Risk of injury and/or failure if the product is not inspected prior to and after use.
- Risk of injury when applying the product outside your field of view.
- Product should be used according to these instructions and with close attention to the safety of patients. Failure to follow these and/or adhere to the healthcare facility's requirements may cause harm to the patient and/or healthcare workers.
- Do not reuse the device when it has been used on a patient with, or suspected of having, Creutzfeldt-Jakob Disease (CJD) or variant CJD (vCJD).
- The product must be inspected, cleaned, and sterilized before each surgical procedure.

Contraindications/Precautions

- Improper use of this product during handling, surgical use or reprocessing, for which they are indicated, may result in damage.
- Do not use excessive stress or misuse as it may result in damage to the product.
- Use only medical detergents to clean this device. Washing with an improper detergent could result in discoloration or corrosion.
- This device is non-sterile
- Material: Radel plastic & stainless steel

Maintenance

Sterilization (minimum parameters)

- Use a validated, properly maintained steam sterilizer. Always follow instructions of the machine manufacturer.
- Do not overload the clip compartments. A clip compartment is considered overloaded when less than 40% of the floor of the compartment is visible when viewed from above.
- Overloading may inhibit steam flow, cause excessive drying time, and make the product too heavy to safely handle.
- Do not exceed 140°C (284°F) during sterilization cycle.
- Do not stack cases. For effective sterilization trays must have adequate steam circulation around all surfaces.
- The product must be placed upright on shelves in order for proper ventilation.
- Do not place the product on their sides or at vertical angles in chamber.
- The product should be wrapped in two layers of 1-ply polypropylene wrap (KC600) or hospital designated FDA cleared sterilization wrapping materials. Always follow the sterilization wrap manufacturer's instruction prior to sterilization.
- Wrapping materials and the trays are designed to allow air removal, steam penetration/evacuation (drying) and maintain the sterility of the internal items.

Effective sterilization can be achieved by following the steam cycles:

Sterilization Method	Pre-vacuum Steam Sterilization	
	Sterilization temp.	Retention time
Sterilization Conditions	134°C/ 273.2°F	3min.
	132°C/ 269.6°F	4min.
	Minimum Dry Time	
	30 min	

Decontamination (minimum parameters)

- Reprocess all devices as soon as reasonably practical following use.
- Follow instructions and warnings as issued by manufacturers of any decontaminants, disinfectants, and cleaning agents used. Wherever possible, avoid use of mineral acids or harsh, abrasive agents.
- Do not use high acid (pH 4.0 or lower) or high alkaline (pH 10 or higher) products for disinfection. Neutral pH detergents 7.0-9.0 are preferred.
- Remove gross contaminants with a steady stream of lukewarm/cool water, not to exceed 35°C (95°F). Rinse each device thoroughly. Do not use saline or chlorinated solutions. Only use distilled or deionized water.
- Do not soak devices in hot water, alcohol, disinfectants or antiseptics solution. Do not exceed 2 hours soaking in any solution.
- Do not use steel wool, wire brushes or abrasive detergents.

Cleaning & Drying: Automated (minimum parameters)

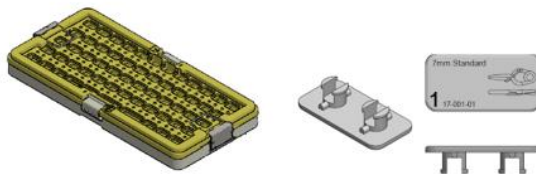
- Use only validated washer-disinfector machines and low-foaming, non-ionizing cleaning agents and detergents.
- Follow the manufacturer's instructions for use, which includes warnings, concentrations and recommended cycles.
- Load the product carefully.
- Ensure that soft, high purified water that is controlled for bacterial endotoxins is used in the final rinse stage.
- Devices must be thoroughly dried and all residual moisture must be removed before they are sterilized. Use a soft absorbent towel or cloth to dry external surfaces.

Cleaning & Drying: Manual (minimum parameters)

- Remove lid and clip tags from tray.
- Rinse the articles under running utility (tap) water to remove gross soil.
- Prepare a detergent bath using Valsure Enzymatic or Valsure Alkaline solution at the manufacturer's recommendation of ½ per gallon using utility (tap) water.
- Fully immerse articles in prepared detergent solution.
- While immersed, brush surfaces with soft bristled brush to remove soil.
- Allow the articles to dwell for 2 minutes.
- Rinse the articles under running reverse osmosis or deionized water for 1 minute.
- Dry articles with clean lint-free cloths.



Inspection and Function Testing

- After cleaning, visually inspect all surfaces for complete removal of soil and fluids. If ANY soil or fluid is still visible, return the device for repeat decontamination.
- Visually inspect and check all devices for damage.
- Remove for repair or replace any worn out, fractured, corroded, stained, discolored or damaged product. Failure to do so may cause injury or harm to the patients or users.


Configuration of Tray Tags

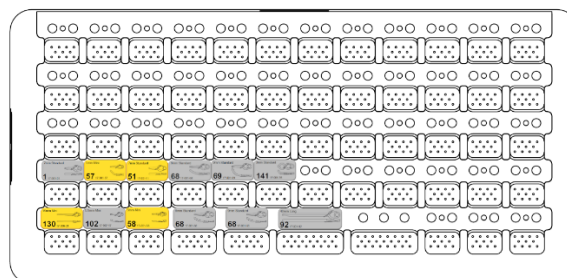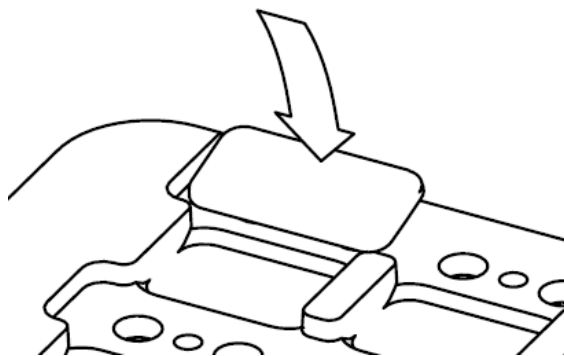- The Tray can be configured with clip identifier tags. These tags identify the clip item number and provide a 1:1 graphic representation of the clip. To apply tags, remove the tray lid, align the tag's locking posts with the connection holes, and press the tag into the tray as shown below:


Re-Configuration of Tray Tags

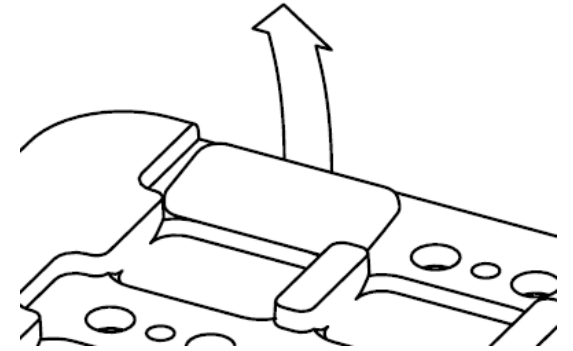- The tags can be removed by grasping on one side and pulling upward. This can be accomplished by pulling upwards on either long edge of the tag, as shown below:

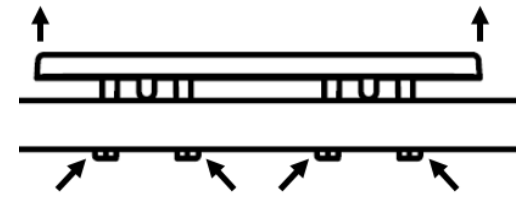- If there is any difficulty removing the tags, you may access the underside of the tray and push inward on the tag's locking posts while simultaneously pulling upward on the tag:

Storage/Life

- Service life of Trays: 7 years
- Always inspect the products between uses to confirm proper functioning.
- Do not store this device in high temperatures or in areas with high humidity where the temperature has drastic variations.
- Non-sterile devices or sterile wrapped devices should be stored in dry, clean conditions at an ambient room temperature.

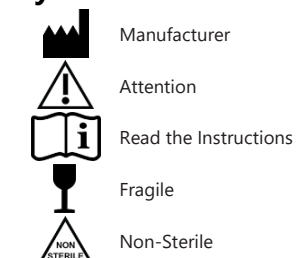
Returns

Note: If the device requires return, the device must be decontaminated. Failure to process prior to return may result in a cleaning charge. Contact Mizuho America via phone (800) 699-CLIP (2547) or email customerservice@mizuho.com to obtain a Returns Goods Authorization (RGA) for return of any items.

Packaging

All devices to be wrapped or packaged following local procedures, in ANSI/AAMI ST 46-1993.

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